

Beau~Belle
Tanning ~ Nails ~ Beauty **spa**

90a South Rd, Waterloo
Liverpool

t: 0151 928 1010



Step 1:
Brush your teeth



Step 2:
Open the foam strip



Step 3:
Take two foam strips and put
them in the mouthpiece using
the provided tweezers.

**COSMETIC
BRIGHT**

" **great results** from our
simple, safe effective
and **fast procedure** "



Step 4:
Look down and put the
mouthpiece in your mouth.
Make sure the foam strips press
firmly to the front side of your
teeth.



Step 5:
Enjoy your session and let
Cosmetic Bright do its work for
20 minutes.



Step 6:
Remove the foam strip, clean
the mouthpiece and put it
back in the hygienic Cosmetic
Bright box.